

# Single-Use Tourniquet

**NEW!**



## The hygienic solution for improved safety

- Minimises risk of contamination
- Easy handling
- No pinching – pain-free also on hairy skin


## Optimal hygienic standard

Single-use tourniquets minimise cross-contamination hazards and nosocomial infections caused by venipuncture.


This tourniquet is made from a special composite material that provides an optimal hygienic standard along with maximum patient comfort.




## User Guide

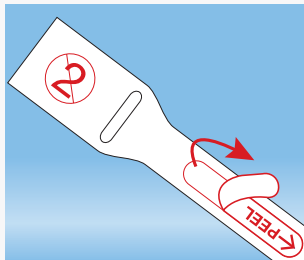


- Pull tourniquet from the packaging and tear off at the perforated line
- Remove label "PEEL HERE"

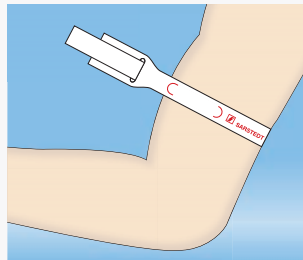


- Pass tourniquet around arm with the printed side up

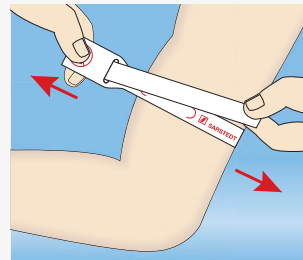

- Fasten a few inches above the puncture site and tape in place
- The pulse must be perceptible (50 - 100 Hg tourniquet pressure)
- Maximum constriction time: 1 min

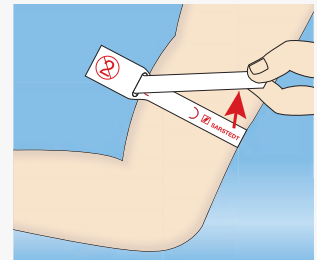

- Peel off and dispose

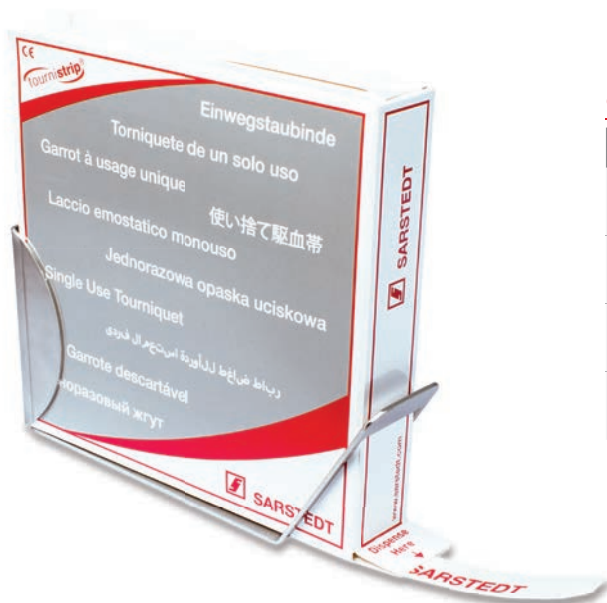

## Ordering information

Order No.	Description	Packaging
95.1006	Single-use tourniquet	200/inner box
95.1006.025	Single-use tourniquet, 25/case	25/inner box
95.1005	Wall holder	1/inner box
95.1004	Paediatric single-use tourniquet	100/inner box

SARSTEDT AG & Co.  
 P.O. Box 12 20  
 D-51582 Nümbrecht  
 Phone +49 2293 3050  
 Fax +49 2293 305-2470  
 info@sarstedt.com  
 www.sarstedt.com